

BIOHIT LACTOSE INTOLERANCE

quick test

QUICK GUIDE


- Allow the test components to reach room temperature before use.
- Biopsy specimen(s) is recommended to be taken from the postbulbar duodenum.
- If there is blood on the specimen, dab the biopsy briefly on a sterile gauze pad.
- Mix all reagents before use by turning the bottles upside down for a few times and tap the bottles on the table to ensure the liquid returns to the bottle.

1. Place the biopsy specimen into the well.

Add two drops of **reagent 1**.

Close the cover of the well and shake the plate 5-6 times horizontally on the table.

Incubate for 15 minutes.

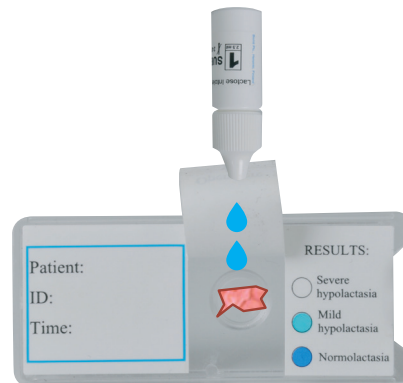

2. Add one drop of **reagent 2** and two drops of **reagent 3**.

Close the cover of the well and shake the plate 5-6 times horizontally on the table.

Incubate for 5 minutes.



3. Read the results against the color scale on the plate or use the interpretation chart included in the product package.



BIOHIT HealthCare

Innovating for Health



DyonMed S.A.
Point of Care Solutions

Λ.Λαυρίου 182—184, 153 54, Παιανία, Αττική.
Τηλ.: 210-6655494 — 495, Fax: 211-7708599
<http://www.dyonmedical.com>
E-mail: customerservice@dyonmedical.com